

Status **Active** PolicyStat ID **19968755**



Origination 08/2004  
Last Approved 02/2026  
Effective 02/2026  
Last Revised 02/2026  
Next Review 02/2029

Owner Sherie Goldbach: Project Coordinator  
Policy Area Infection Prevention  
Applicability UNC Medical Center  
Policy Tag Groups Infection Prevention

## Endoscope

### I. Description

Describes the steps for cleaning, high-level disinfection (HLD), and sterilization of endoscopes.

### II. Policy

#### A. Staff Should Adhere to Guidelines in the Following Infection Prevention Policies

1. [Exposure Control Plan for Bloodborne Pathogens](#)
2. [Hand Hygiene and Use of Antiseptics for Skin Preparation](#)
3. [High-Level Disinfection \(HLD\) - Manual Reprocessing of Reusable Semi-Critical Medical Devices](#)
4. [Sterilization of Reusable Patient-Care Items](#)

#### B. HLD Competency and Training

1. If processing Endoscopes using HLD, please refer to the guidance in the Infection Prevention policy: [High-Level Disinfection \(HLD\)- Manual Reprocessing of Reusable Semi-Critical Medical Devices](#).

- Staff who perform HLD activities are required to attend the initial HLD class as soon as possible after assignment of HLD responsibilities. All work should be supervised by a competent staff member until competency is documented for each reprocessing task.
2. Please see the [Infection Prevention's Instrument Reprocessing intranet site](#) for the most current competency form for the HLD processes performed in your specific area and specific devices.
  3. Initially and annually, four elements must be completed in order to meet UNC Hospitals' requirements for HLD competency:
    - a. Training
      - i. Initially, complete the HLD Class (registration through LMS) and receive on-the-job practice with a competent person.
      - ii. Annually, complete the HLD Refresher Course education in LMS and contact Infection Prevention with any questions.
    - b. Demonstration
      - Able to perform HLD with no assistance.
    - c. Observation
      - Competent person observes your entire process.
        - Competency testing should be conducted by staff with a valid competency form on file at commencement of employment and at least annually thereafter. The approved competency form for UNC Hospitals facilities is available on the [Infection Prevention's Instrument Reprocessing intranet site](#).
    - d. Documentation
      - Competent person completes your competency form.
        - Completed competency forms will be kept at an immediately accessible location.
  4. All HLD staff must take an LMS module on color blindness.
  5. Temporary personnel should not be allowed to reprocess endoscopes until competency has been established.
  6. Staff who infrequently reprocess endoscopes (e.g., once every 3 or more months) will require review and may need repeat competency testing more frequently than annually.
  7. Key personnel who are involved in staff education of cleaning/disinfection processes must have documented training by the company representatives prior to introduction of new

endoscope being used or processed. These key personnel may now educate other staff who have the responsibility of teaching reprocessing of endoscopes to others in their area.

8. For guidance on area-specific questions, Infection Prevention should be consulted.

## **C. Summary – The Seven Steps for High-Level Disinfection of Flexible Endoscopes**

1. For all steps of HLD, staff will don appropriate personal protective equipment (PPE).
2. It is strongly recommended to use disposable buttons, brushes, and endotherapy instruments.
3. Refer to scope manufacturer's instructions for use (MIFU) when damaged scopes are reported.
4. Endoscopes should have periodic in-depth inspections by the manufacturer per the MIFU.
5. Cleaning the Endoscope and Accessories in the Reprocessing Area.
  - a. Only personnel who have been trained and competency tested may process endoscopes and their accessories. Refer to the section Training and Competency Testing.
  - b. Staff involved in endoscope reprocessing should adhere to standard infection prevention and control recommendations to protect both patient and staff.
  - c. Supplies and equipment:
    - i. Personal protective equipment (PPE), including but not limited to extended cuff gloves, full face protection to include eyes, nose, and mouth, impervious gown, and safety glasses/goggles, must be worn.
    - ii. Leak testing equipment.
    - iii. Channel cleaning adapters per both the endoscope and automated endoscope reprocessor (AER) MIFU.
6. Pre-Cleaning: Preparing the Endoscope for Cleaning
  - a. Pre-cleaning of flexible endoscopes and reusable accessories is done at point of use by following the scope MIFU. Pre-cleaning may include the use of an enzymatic detergent. Perform immediately following completion of the endoscope procedure to help prevent the formation of biofilm.
  - b. Transport the scope and accessories to the reprocessing area in a leak-proof container with a Biohazard label on it. Place "dirty" tag on the grip section of the scope.
  - c. Scopes needing reprocessing should be placed in a designated area used for dirty scopes only.

## 7. Leak Testing

- a. Perform pressure/leak testing after each use and before formal reprocessing according to MIFU.
- b. If a leak is detected or the endoscope appears damaged, follow the MIFU for cleaning a damaged scope. Follow the automated endoscope reprocessor (AER) instructions for AER able to high-level disinfect damaged scopes.

## 8. Cleaning may occur via Manual Methods or via Validated Automated Endoscope Reprocessing Cleaning Step.

- Manual cleaning
  - i. Should occur within the manufacturer's recommended time frame. Devices and instruments should be kept moist between use of an instrument/device and the start of treatment to prevent soils from drying, unless prohibited/incompatible per the MIFU.
  - ii. Meticulous manual cleaning is important to remove the microbial burden from endoscopes, accessories, and instruments. Follow the endoscope MIFU for cleaning. Include valve covers, channels, elevator channel assemblies, connectors, and all detachable parts. Retained debris may adversely affect the efficacy of the high-level disinfectant (HLD).
  - iii. Follow MIFU for both the enzymatic detergent and endoscope.
  - iv. Use fresh enzymatic cleaner solution for each endoscope to prevent cross-contamination.
  - v. Discard disposable parts. The endoscope must be completely disassembled so all surfaces may be reached for thorough cleaning.
  - vi. Use a digital timer to time soak duration.
  - vii. Use brushes appropriate for the size of the endoscope channel, parts, connectors, and orifices.
  - viii. It is strongly recommended that only single-use, disposable channel brushes be used. If reusable brushes must be used, the brush must follow the same disinfection procedure as the scope after each use of the reusable brush.
  - ix. When applicable, attach automated endoscope flushing aid (e.g., Scope Buddy™) for the type of scope being cleaned. Proceed per MIFU.
  - x. For areas without an automated endoscope flushing aid, follow MIFU for cleaning the specific scope make and model.
  - xi. Reusable endoscopic/endotherapy instruments (e.g., biopsy forceps and other cutting instruments) that break the mucosal barrier should be

cleaned and sterilized between each patient use following MIFU.

- xii. Ultrasonic cleaning of reusable endoscopic accessories and endoscope components may be used to remove soil and organic material from hard-to-clean areas.
  - xiii. Certain models of endoscopes (i.e., duodenoscopes) require specialized cleaning and brushing of elevator channels. Refer to scope MIFU.
  - xiv. Automated flushing aid attachments facilitate cleaning with forced flow of fluids.
    - The exterior must be manually rinsed with water to remove residual debris and enzymatic cleaning.
    - Dry the endoscope and accessories with a soft, lint-free cloth to prevent dilution of the high-level disinfectant if manually disinfecting.
  - xv. For areas without an automated flushing aid attachment
    - Thoroughly rinse the exterior of the endoscope, all channels and lumens, and all removable parts with copious warm running water to remove residual debris and enzymatic cleaner.
    - Dry the endoscope and accessories with a soft, lint-free cloth to prevent dilution of the high-level disinfectant if manually disinfecting. Follow AER MIFU regarding drying the endoscope prior to placing in the AER.
- Validated Cleaning with the Automated Endoscope Reprocessor Steps
    - i. For Automated Endoscope Reprocessors that have validated cleaning steps (e.g., Medivator Advantage Plus, EVOtech, TE100, Astra) as part of their MIFU, this step may replace manual cleaning when used according to the MIFU of the AER and the device.
    - ii. Multidisciplinary team will review and determine when automated cleaning is used as an appropriate alternative to manual cleaning.

## 9. Visual Inspection

- a. Visually inspect the endoscope and its accessories before high-level disinfection occurs.
- b. Damage to the endoscope can prevent adequate cleaning and disinfection. If there are concerns for damage, the manufacturer should be contacted to determine if more detailed inspection should occur by the manufacturer.

## 10. High-Level Disinfection (HLD)

- a. Following cleaning and visual inspection, perform HLD or sterilization in accordance with the MIFU. Carefully review and adhere to the endoscope manufacturer's reprocessing instructions and to the MIFU for chemicals or sterilants and any equipment (e.g., automated endoscope reprocessors) used for reprocessing to help ensure that effective disinfection occurs.
- b. Endoscopes and accessories that come in contact with mucous membranes are classified as semi-critical and should receive at least high-level disinfection after each patient use.
- c. Use high-level disinfectants cleared by the FDA and compatible with the endoscope per MIFU.
- d. Comply with all applicable safety policies regarding exposure monitoring to minimize personnel exposure.
- e. Prepare and verify minimum effective concentration of the high-level disinfectant according to the MIFU.
- f. Containers or basins used during manual HLD should be washed with enzymatic detergent and water, rinsed, and dried on a regular basis, e.g., daily for enzymatic detergent containers/basins, and when the high-level disinfectant expires, or fails minimum effective concentration.
- g. High-level disinfectants must be approved by Infection Prevention prior to purchase. Refer to the Infection Prevention policy: [High-Level Disinfection \(HLD\) - Manual Reprocessing of Reusable Semi-Critical Medical Devices](#) for approved high-level disinfectant chemicals.
- h. Special Considerations for Manual High-Level Disinfection (HLD)
  - i. Completely immerse the endoscope, all removable parts, accessories, and instruments in a basin of properly prepared HLD solution. The basin must be compatible with the high-level disinfectant and of a size to accommodate the endoscope without undue coiling. HLD containers must have a tight-fitting lid to contain the vapors of the high-level disinfectant. Lids must be kept on the containers except when placing an endoscope into and taking an endoscope out of the disinfectant. This helps contain vapors. The reprocessing area should have engineering controls to ensure good air quality.
  - ii. To prevent damage to the endoscopes, scopes must be soaked one per container and never crowded. Do not soak any sharp instruments with the endoscope that may potentially damage the endoscope.
  - iii. Soak the endoscope, instruments, and accessories in the high-level disinfectant for the specified time/temperature required to achieve high-level disinfection per the chemical MIFU. Use a timer to verify soaking

time.

- iv. Removable parts should be disinfected simultaneously.
- v. Flush all channels completely with air before removing the endoscope from the high-level disinfectant. This preserves the concentration and volume of high-level disinfectant and prevents exposure from dripping and spilling.

#### 11. Rinse After Manual Disinfection

- Thoroughly rinse the exterior surfaces of the insertion tube with tap water. Suction or perfuse copious amounts of water through the biopsy and air/water channels to rinse all disinfectant from the interior surfaces of the scope. Discard the rinse water after each use/cycle.

#### 12. Drying

- a. Flush all channels, including accessory channels, with alcohol until the alcohol can be seen exiting the opposite end of each channel.
  - i. Alcohol purges should be used even when sterile water is used for rinsing.
  - ii. 70-90% ethyl or isopropyl alcohol is used as a solvent to assist drying the interior channel surfaces.
- b. Dry all channels with instrument air. Avoid the use of excessively high air pressure, which may damage the internal channels of the flexible endoscopes. Review MIFU for ideal and maximum psi to be used. The length of channels in gastrointestinal endoscopes may require an extended drying time.
- c. Remove all channel adapters if used.
- d. Dry the exterior of the endoscope with a soft, clean, lint-free towel.
- e. Thoroughly rinse and dry all removable parts. Do not attach removable parts (valves, etc.) to the endoscope during storage.

#### 13. Storage and Management of Scopes

- a. Staff reprocessing endoscopes must ensure that users can readily identify that the endoscope has been reprocessed. Endoscopes ready for use are tagged with a "clean" tag. If a scope does not have a "clean" tag attached, the scope must not be used on a patient.
- b. Endoscopes should be stored in a manner that will protect the endoscope from contamination and damage.
  - To facilitate drying, the endoscope should be stored vertically with the distal tip hanging freely without a tip protector.

- c. Do not store in the carrying case. Carrying cases supplied by the manufacturer are not suitable for storage.
- d. Clean gloves should be worn when handling processed/clean endoscopes.
- e. Once a scope leaves the clean storage cabinet, regardless of whether it is used on a patient, it should be reprocessed per MIFU.

#### 14. Documentation

- Maintain documentation for each step every time an endoscope is reprocessed. Logs are posted on the [Infection Prevention's Instrument Reprocessing intranet site](#)

## D. Automated Reprocessing

1. Automated endoscope reprocessors (AER) standardize the disinfection process and decrease personnel exposure to disinfectants.
2. Follow all steps for the mechanical cleaning of the endoscope before using an automated reprocessor. For an AER that is FDA validated to perform the cleaning process, follow the AER MIFU for automated cleaning.
3. Compare the MIFU provided by both the endoscope and the AER manufacturer and resolve any conflicting recommendations.
4. The elevator channel assembly of duodenoscopes must be processed according to the scope's MIFU and AER MIFU.
5. Automated reprocessors must receive preventive maintenance by qualified personnel, and this should be guided by AER MIFU.
6. Prepare and use the AER according to MIFU.
7. Use correct adapters according to MIFU.
8. Place valve covers and other removable parts into the appropriate container in the basin of the reprocessor.
9. Start the machine and allow it to complete all cycles/phases. If cycles/phases are interrupted, high-level disinfection cannot be ensured.
10. If a final alcohol rinse cycle is not included in the automated reprocessor, this step should be done manually.
11. Dry and store as described in manual disinfection.
12. If the automated endoscope reprocessor produces a printout, those printouts shall be kept on file for 5 years. For printer paper that fades, scan the printouts into a computer file that is accessible to staff.
  - Staff removing the scope from the AER must initial the printout immediately after

each cycle finishes.

## **E. Disinfection Of Water Bottles and Use of Disposable Water Bottle Systems Which Provide Water Via The Endoscope During Procedures**

1. If a disposable tubing system and disposable water bottle are used, follow MIFU and disposal.
  - A new disposable water bottle/tubing system must be used before every ERCP procedure. Sterile water should be used to fill the water bottles.
2. If reusable water bottles are used, they must be washed, disinfected, and sterilized at the end of each day and/or before an ERCP procedure.

## **F. Flexible Lumened Endoscope Tracking**

- Logs/records for flexible lumened endoscopes should include the patient's identifier (i.e., MRN, CSN), date of procedure, and a scope identifier (e.g., serial number or other unique identifier). Document the endoscope's unique identifying number in the electronic medical record of the patient. This system allows for follow-up should there be a problem with the scope or processing procedures. Logs should be maintained for 5 years.

## **G. Additional Instructions for Duodenoscopes (ERCP Scopes)**

- Flexible duodenoscopes must be double high-level disinfected after each use. It is not necessary to decontaminate the scope via the enzymatic detergent and brushing steps before the second run through the AER. The first AER cycle should be completed, printouts checked and initialed, and the cycle run a second time, and printouts checked and initialed as usual.

## **H. High-Level Disinfection of Non-Lumened Flexible Endoscopes**

1. Pre-clean the endoscope per MIFU.
  - Transport the scope to the area designated for cleaning dirty scopes in a covered container marked biohazard. Protect the scope by supporting the insertion tube and body of the scope.
2. Perform a leak test after each use. Follow the MIFU.
3. Mechanically clean the scope according to MIFU.
4. Dry the endoscope and accessories.

## 5. High-Level Disinfection

- After cleaning the scope, following MIFU, immerse the scope in the HLD chemical, using a basin large enough to allow for total immersion and cover with a tight-fitting lid.
6. Rinse the endoscope thoroughly with a large amount of clean water.
  7. Dry the endoscope with a soft, lint-free cloth or towel.
  8. Rinse or wipe the endoscope with 70% ethyl or isopropyl alcohol.
  9. Store and manage the scopes in a clean manner in an area/cabinet designated for clean endoscopes.
    - a. If a water-resistant cap is used for processing these scopes, it should be left off during storage to allow the endoscope to breathe.
    - b. If storing in a scope cabinet, the distal end of the scope should not touch the bottom of the cabinet.
    - c. Once a scope leaves the clean storage cabinet, regardless of whether it is used on a patient, it should be reprocessed per MIFU.
    - d. Clean gloves should be worn when handling processed/clean endoscopies.

## I. Rigid Endoscopes

Endoscopes that normally pass through sterile tissues (e.g., arthroscopes, laparoscopes) are critical devices and should be sterilized after each use and maintained in a sterile condition via an appropriate wrap or tray until use.

1. Follow MIFU for cleaning, rinsing, and drying prior to disinfection or sterilization.
  - a. Thorough cleaning, including exterior and interior channels and sheaths, must be performed prior to disinfection or sterilization.
    - i. Non-integral parts (e.g., biopsy valve and tip cover) of the endoscope should be removed, according to the manufacturer's recommendations, and processed separately.
    - ii. Any enzymatic surgical instrument cleaning solution may be used with sufficient scrubbing action to remove gross soil.
    - iii. Leak testing is not performed on rigid endoscopes.
    - iv. After cleaning the exterior and interior channels, sheaths must be rinsed and then dried as much as possible before immersing into a HLD liquid or placed into a sterilization container.
  - b. High-level disinfection can be achieved by completely submerging the endoscope in disinfectant solution with all trapped air removed. Using appropriate cleaning

adapters such as syringes, fill all channels of the endoscope with disinfectant until it can be seen exiting the opposite end of each channel. Ensure that all channels are filled with high-level disinfectant and that no air pockets remain within the channels. Pressure lock by leaving the syringe attached to the channel to retain the disinfectant for the amount of time recommended by the manufacturer to ensure complete microbial destruction.

- c. Thoroughly rinse the exterior surfaces of the insertion tube with tap water. Suction or perfuse copious amounts of water through the biopsy and air/water channels to rinse all disinfectant from the interior surfaces of the scope. Discard the rinse water after each use/cycle.

## 2. Drying High-Level Disinfected Rigid Endoscopes

- a. Flush all channels, including accessory channels, with alcohol until the alcohol can be seen exiting the opposite end of each channel.
  - i. Alcohol purges, with 70-90% ethyl or isopropyl alcohol, should be used even when sterile water is used for rinsing.
  - ii. Use fresh alcohol that has been properly stored in a closed container between uses.
- b. Dry all channels with instrument air.
- c. Remove all channel adapters if used.
- d. Dry the exterior of the endoscope with a soft, clean, lint-free towel.
- e. Thoroughly rinse and dry all removable parts. Do not attach removable parts (valves, etc.) to the endoscope during storage.

## 3. Storage: Staff reprocessing endoscopes must ensure that users can readily identify whether and when an endoscope has been reprocessed. Endoscopes ready for use are tagged with a "clean" tag.

- a. Endoscopes should be stored in a manner that will protect the endoscope from contamination and damage.
- b. A storage cabinet with adequate ventilation will encourage continued air drying of the surfaces and prevent undue moisture buildup in the cabinet interior.
  - Scope storage cabinets need to be maintained in a clean manner, with no dust/debris on the bottom, and should be cleaned on a regular basis.
- c. If a cabinet is not available, the endoscope may be stored on the endoscope cart or other designated area in a clean location.
- d. Do not store in the carrying case.

# J. Sterilization of Endoscopes, Accessories, and Ancillary Equipment Used Internally

## 1. General Information

- a. Instruments that contact sterile tissues or the vascular system must be sterile. Examples include endoscopes passing through a surgical opening, biopsy forceps, and cytology brushes.
- b. Clean, rinse, and dry per the Infection Prevention: [Sterilization of Reusable Patient-Care Items](#).
- c. The instruments/accessories to be sterilized should be placed in an individual sealed pack, such as a peel pack.

## 2. Methods of Sterilization – Endoscopes

- a. Steam sterilization should be used when indicated by scope MIFU.
  - The scope's MIFU for the sterilization process must be followed.
- b. Hydrogen peroxide gas plasma is generally not used for endoscopes because of restrictions on lumen diameter and length.

## 3. Sterilization – Accessories and Ancillary Equipment Used Internally

### • General Information

- i. Biopsy forceps, cytology brushes, and other cutting instruments that contact the vascular system must be sterilized. Prior to sterilization, these items must be cleaned according to the device MIFU.
- ii. Packaging and labeling of items for sterilization.
  - Items to be sterilized should be prepared and packaged so sterility can be achieved and maintained to the point of use.
  - Sterilized items should be labeled with a load number that indicates the sterilizer used, the cycle or load number, and the date of sterilization.
  - Sterile packages should be evaluated before use for loss of integrity (e.g., torn, evidence of moisture, punctured).
  - Sterilized packs may be used (unless labeled with an expiration date) unless the integrity of the packaging is compromised.

## 4. Equipment not used internally, such as power sources, illuminators, insufflators, and suction devices, should be cleaned according to MIFU.

## K. Environmental Services

- Cleaning and disinfection of reprocessing environment is performed daily with an EPA-registered disinfectant cleaning solution.

## L. Safety

1. Appropriate personal protective equipment (PPE) should be readily available and used to protect workers from exposure to chemicals or microorganisms (e.g., HBV).
2. Facilities where endoscopes are used and disinfected should be designed to provide a safe environment for staff and patients.
  - Air-exchange equipment (ventilation system, exhaust hoods, etc.) should be used to minimize the exposure of all persons to potentially toxic vapors as indicated in the Safety Data Sheet.
3. Infection Prevention will conduct rounds in reprocessing areas to ensure that reprocessing instructions are current, accurate, and correctly implemented.

## M. Quality Assurance and Surveillance

1. ERCP scopes are cultured using the CDC-specified guidelines for quality improvement purposes only. Cultures are not meant to monitor adequate cleaning and high-level disinfection of ERCP scopes.
2. If a cluster of endoscopy-related infections occurs or is suspected, an investigation will be initiated to determine the potential routes of transmission and reservoirs. Notify Infection Prevention (984-974-7500) for assistance with any questions or problems.

## N. Implementation

The Medical Director and supervisor of each department or his/her designee will identify persons responsible for supervising the implementation of this policy.

## III. References

Rutala, WA. APIC guidelines for selection and use of disinfectants. *Am J Infect Control* 1996; 24:313-342.

Multisociety guideline on reprocessing flexible GI endoscopes: 2016 update. *Gastrointestinal Endoscopy*. Volume 93. No. 1. 2021

Rutala, WA; DJ Weber and The Healthcare Infection Control Practices Advisory Committee. CDC Guidelines for Disinfection and Sterilization in Healthcare Facilities, *MMWR*. 2008

Healthcare Infection Control Practices Advisory Committee (HICPAC). Essential Elements of a Reprocessing Program for Flexible Endoscopes - Recommendation of the HICPAC. Centers for Disease

Control and Prevention. Retrieved 02/2021

APIC Text. Chapter 56: Endoscopy. Association for Professionals in Infection Control and Epidemiology. Retrieved 02/2021

Food and Drug Administration - Duodenoscope Surveillance Sampling & Culturing: Reducing the Risks of Infection. <https://www.fda.gov/media/111081/download>. Retrieved 03/2021

## IV. Related Policies

[Environmental Health and Safety Policy: Personal Protective Equipment Requirements](#)

[Infection Prevention Policy: Exposure Control Plan for Bloodborne Pathogens](#)

[Infection Prevention Policy: Hand Hygiene and Use of Antiseptics for Skin Preparation](#)

[Infection Prevention Policy: High-Level Disinfection \(HLD\) - Manual Reprocessing of Reusable Semi-Critical Medical Devices](#)

[Infection Prevention Policy: Sterilization of Reusable Patient-Care Items](#)

[Occupational Health Services Policy: Infection Prevention and Screening Program: Occupational Health Service](#)

## V. Responsible for Content

Infection Prevention



### All Revision Dates

02/2026, 04/2024, 06/2022, 01/2022, 06/2021, 08/2019, 07/2018, 10/2016, 04/2015, 05/2012, 05/2009, 09/2006, 08/2004

### Approval Signatures

Step Description	Approver	Date
Policy Stat Administrator	Judith Strubin: Mgr Program-IP	02/2026
	Emily Vavalle: HCS Exec Dir Infection Prevention	02/2026
	Sherie Goldbach: Project Coordinator	02/2026

---

## Applicability

UNC Medical Center

COPY